

# MAINTENANCE AND WARRANTY GUIDE

## AQUAROLL® A2S1 SYSTEM



### INITIAL INSTALLATION

Professional installation of the Aquaroll® A2S1 system is highly recommended. Service and installation contracts are available, upon request.

- **Caution:** Glass beads must be thoroughly rinsed with one (1) gallon of water for one (1) minute prior to initial client use of the Aquaroll® A2S1 system.
- **Caution:** Only use glass beads supplied by Aquaroll®, as they are specially designed for full functionality.
- **Caution:** Do not exceed two and a half (2.5) bags of glass beads at one time in the system.
- **Note:** Fewer beads may be used when treating smaller extremities. The maximum quantity of beads may be utilized when treating large feet, such as with male athletes.

### STORAGE REQUIREMENTS

Always press the “On/Off” power switch to “Off” and unplug the device after each daily usage session.

Store the Aquaroll® A2S1 system in a moderate, dry location.

- **Caution:** Never store the Aquaroll® A2S1 system in a location with a temperature below fifty (50) degrees Fahrenheit.
- **Caution:** Never allow unattended children or mentally disabled clients around the Aquaroll® A2S1 system, as there is a risk of ingestion of glass beads.

It is recommended to cover the device, when not in use.

Where necessary to discard the Aquaroll® A2S1 system, ensure full compliance with local waste regulations.

### MAINTENANCE

The drainage valve of the tank must be cleaned quarterly, at a minimum. Refer to the Aquaroll® Care and Cleaning Instructions (90000007) for further details.

Glass beads require replacement after every thousand (1,000) treatment sessions. In the case of high-volume usage, this equates to every one to two (1-2) months.

- **Caution:** Failure to replace the glass beads can result in decreased client benefit, due to the loss of uniformity and constancy of the percussive hydro-massage. Such failure can also result in worn-down beads getting stuck inside the drainage and system tubing and can void the warranty.

### WARRANTY INFORMATION

The Aquaroll® A2S1 system comes with a one (1) year warranty on parts.

**The warranty is void upon the following conditions:**

- **improper use and/or evidence thereof**
- **attempted opening and/or repair of the machine by anyone other than the designated service professional**
- **loss of the original packaging**

**The parts warranty is not valid where the Aquaroll® A2S1 system is returned in packaging other than the original shipment packaging.**

Additional parts and labor warranties are available for purchase, upon request. Please contact AM Rejuvenation for details.

---

#### AM Rejuvenation, LLC

6538 Collins Avenue, #313  
Miami Beach, FL 33141

To order or for info, call:  
305.504.9690

Find more info online:  
[www.aquarollUSA.com](http://www.aquarollUSA.com)

Request info by email:  
[sales@aquarollUSA.com](mailto:sales@aquarollUSA.com)